

ALLFIT Training Material

Training Manual



OxygenCeuticals
breathe life into your skin

ALLFIT

Non Invasive Dual Skin Remodeling System

Combined technology of OXY-FU & OXY-FX
for noninvasive, fundamental skin remodeling.



OXY-FU (High intensity focused ultrasound) : Elasticity, wrinkle, facial contouring care
OXY-FU technology that uses 7MHz of focused high intensity ultrasound.

OXY-FX (Local Dynamic Micro-Massage) : Collagen & Elastin production/Skin lifting
OXY-FX technology where multiple frequencies (1,3,10,17MHz) can be selectively used on desired layer of skin.

Customized Therapy

OXY-FU and OXY-FX, dual technologies in 1 platform,
convenient to use for various therapies.

Economical Value

The OXY-FU cartridge can be used up to **500,000** shots,
making it cost beneficial.

(Avg. 1person treatment : 1,500~2,000 shots

One cartridge lasts for up to 100~120 treatments)

Multiple Frequencies

Differentiated synergistic effect by releasing up to 4 different
ultrasound frequencies (1,3,10,17MHz).

Simple UI

Uses intuitive Graphical User Interface (GUI),
making it user friendly.

OXY-FU

'High-Intensity Focused Ultrasound'

By irradiating strong ultrasonic energy, it forms a thermal coagulation point on a specific area of deep tissue, resulting in the instant lifting of skin without damaging the epidermis.

- 50~65°C heat occurrence in skin tissue (Coagulative Necrosis)
- Induces collagen and elastin fiber resynthesis through contraction of specific tissues.
- Provides natural lifting & improved elasticity through non-surgical method.



1) OXY-FU Handpiece



Ergonomically Designed Handpiece

Pen-type handpiece allows delicate treatment on all areas of the face. The ALLFIT handpiece gives no pressure on hands with its easy grip design.

Automatic Continuous Irradiation Function

ALLFIT has an automatic continuous irradiation function installed in its system, allowing for effective facial lifting through just a one time program set up.




Guaranteed High Safety

Lower chance of machine malfunction through replacement of handpiece as a whole.


2) OXY-FU Mechanism

True lifting through accurate targeting


OXY-FU delivers high-intensity focused ultrasonic energy to different skin layers, resulting in natural lifting and improved elasticity.

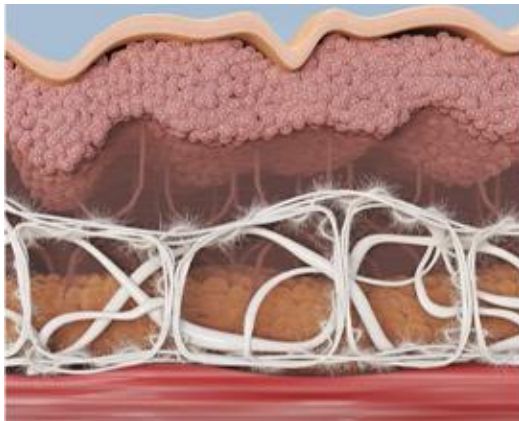

01. Dermal layer

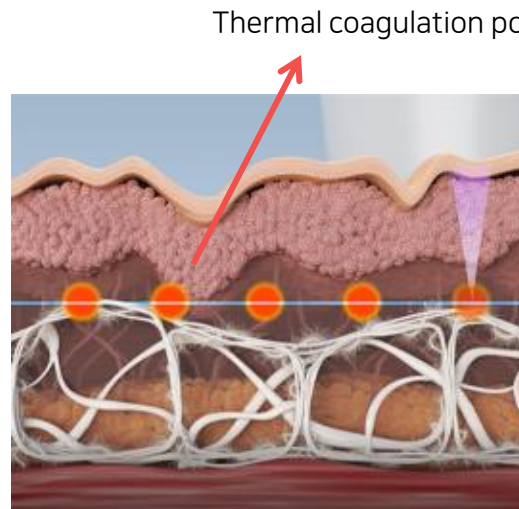

02. Heat coagulation formed through precise targeting

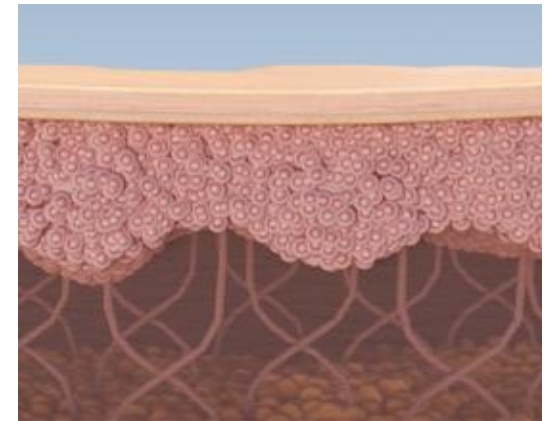

03. Stimulation of collagen regeneration/contraction

3) OXY-FU Benefits

Non-Invasive

Versatile lifting device enables possibility to treat the deepest layer.

OXY-FU provides facial lifting by improving skin elasticity through non-invasive means.


Wide range treatment options targeting at different layers

Low pain, exact targeting treatment using precise frequency matching technology.

Offers wide range of options for customized treatment.

Standard inclusion: 3.0mm handpiece.

Optional add on handpiece: 1.5mm, 4.5mm & 6.0mm

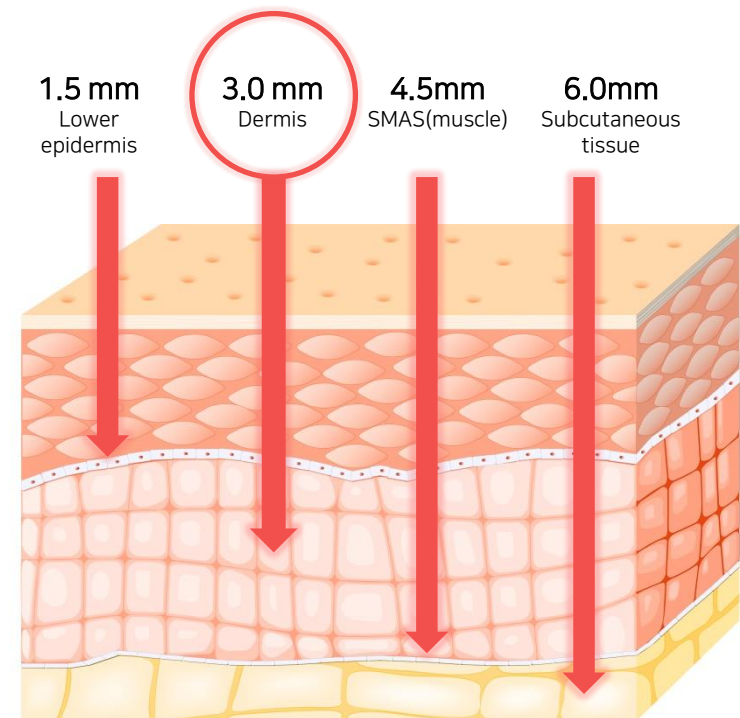


4) OXY-FU Usage Guide (by facial area)

- The average shot usage per treatment is 1,500~2,000 shots for the entire face.

※ Adjust Power/Interval depending on skin type and treatment area.

※ Power : Strength of shot
Hz : Speed of shot released

※Caution※
Do not use on
eye and thyroid
area

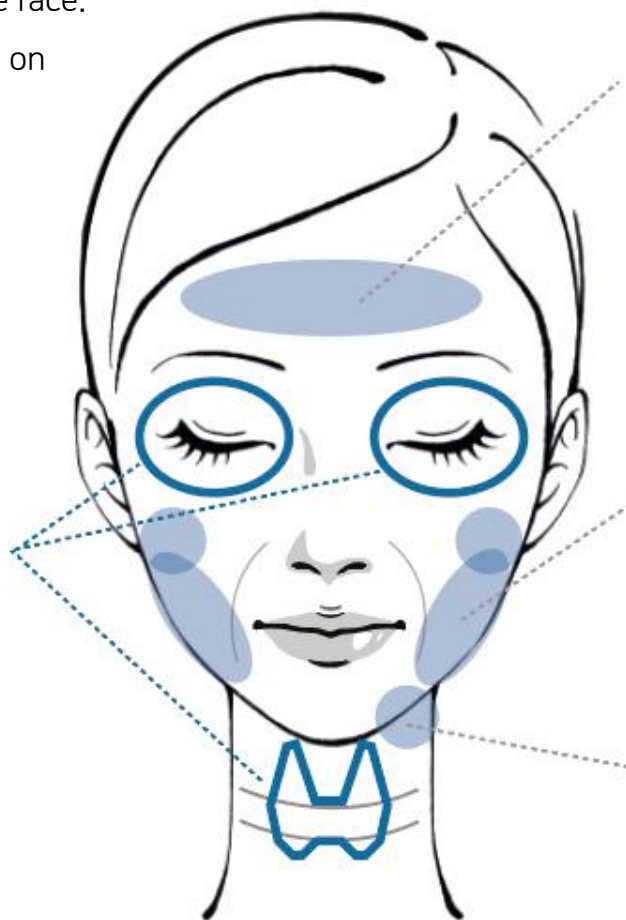

1. Above eyebrows/forehead line

Power : Level 1~2
Interval : 5Hz

2. Chin and jaws (left/right)

Power : Level 3~5
Interval : 5Hz

3. Chin (left/right)

Power : Level 3~5
Interval : 5Hz

OXY-FX

Localized Dynamic Micro-massage/wave

Provides solution to complex skin problems by maintaining balance within the extracellular matrix (ECM) using the modulation stimulation of 4 different frequencies (1,3,10,17MHz).

- Ultrasound treatment of up to 4 frequencies available (5 programs for different skin types)
- Suitable for pre/post clinical treatment care
- Treatment of various skin concerns (skin aging, acne etc.)
- Skin treatment through the optimization of regenerative immune system.




1) OXY-FX Mechanism

' All in one! Max 4 Frequency Treatment '

ALLFIT handpiece releases up to 4 frequencies (1,3,10,17MHz) in 1/500 second intervals, creating a new wavelength and dynamic pressure gradient within the skin.

This provides overall solution to skin concerns by balancing cell synthesis and degradation.


Skin Penetration Depth by Frequency

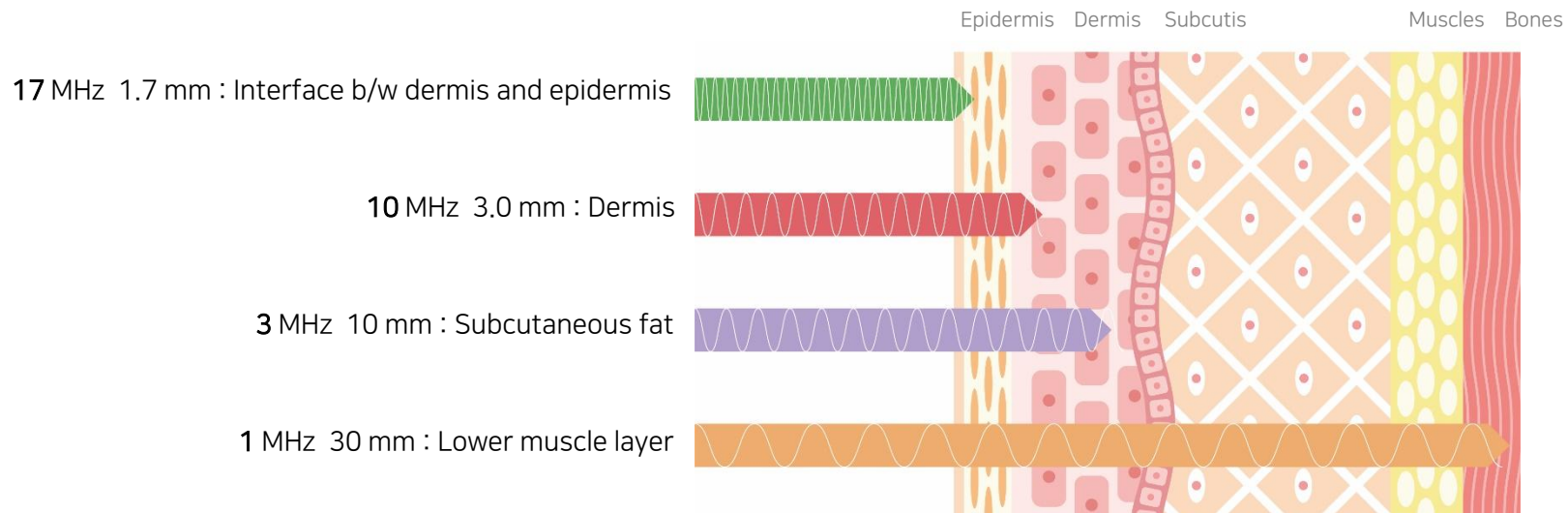

2) OXY-FX Mechanism

Advanced Technology : 17MHz


ALLFIT includes 17MHz frequency in addition to the 3 commonly used frequencies: 1,3,10MHz. The 17MHz effectively targets the layer of the skin that is closely related to skin aging and changes, allowing more effective treatment.

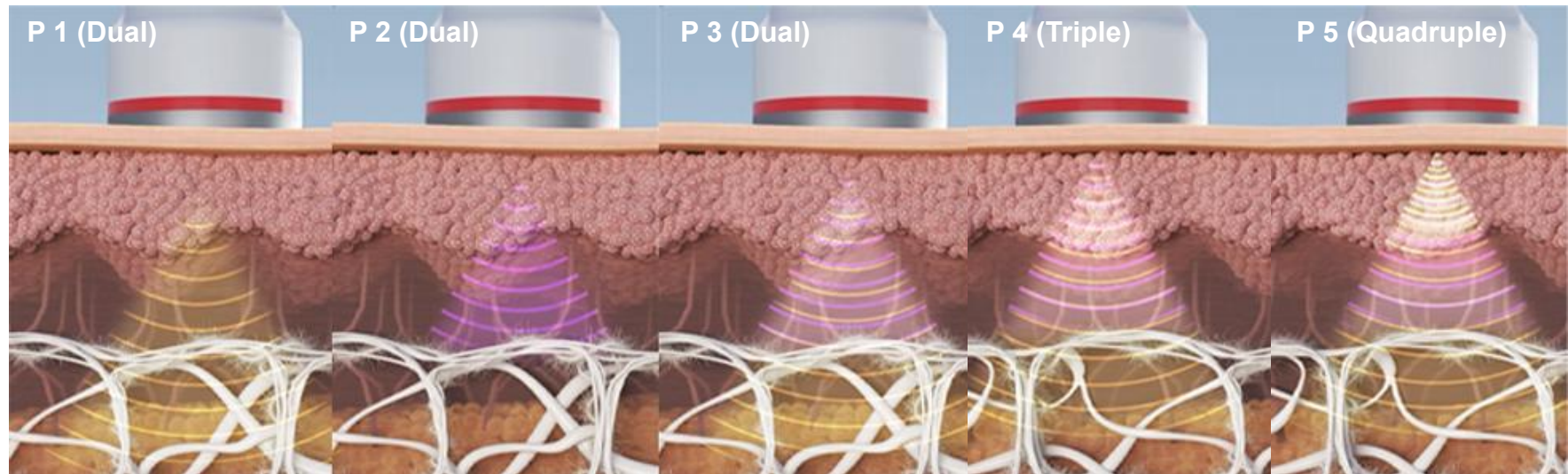

- **P1** (1,3MHz) : Deepest penetrations, square jaw, double chin
- **P2** (3,10MHz) : Fine lines around eyes, etc., deep acne care
- **P3** (10,17 MHz) : Water glow, pore management, surface acne care
- **P4** (1,3,10 MHz) : Deep wrinkles, pore management
- **P5** (1,3,10,17MHz) : Complex skin problems, pre/post procedure care treatment

3) OXY-FX Usage Guide (by facial area)

- Recommended usage for entire face is 10 minutes.
Treatment should not exceed 20 minutes.
- Use 1–2 per week.

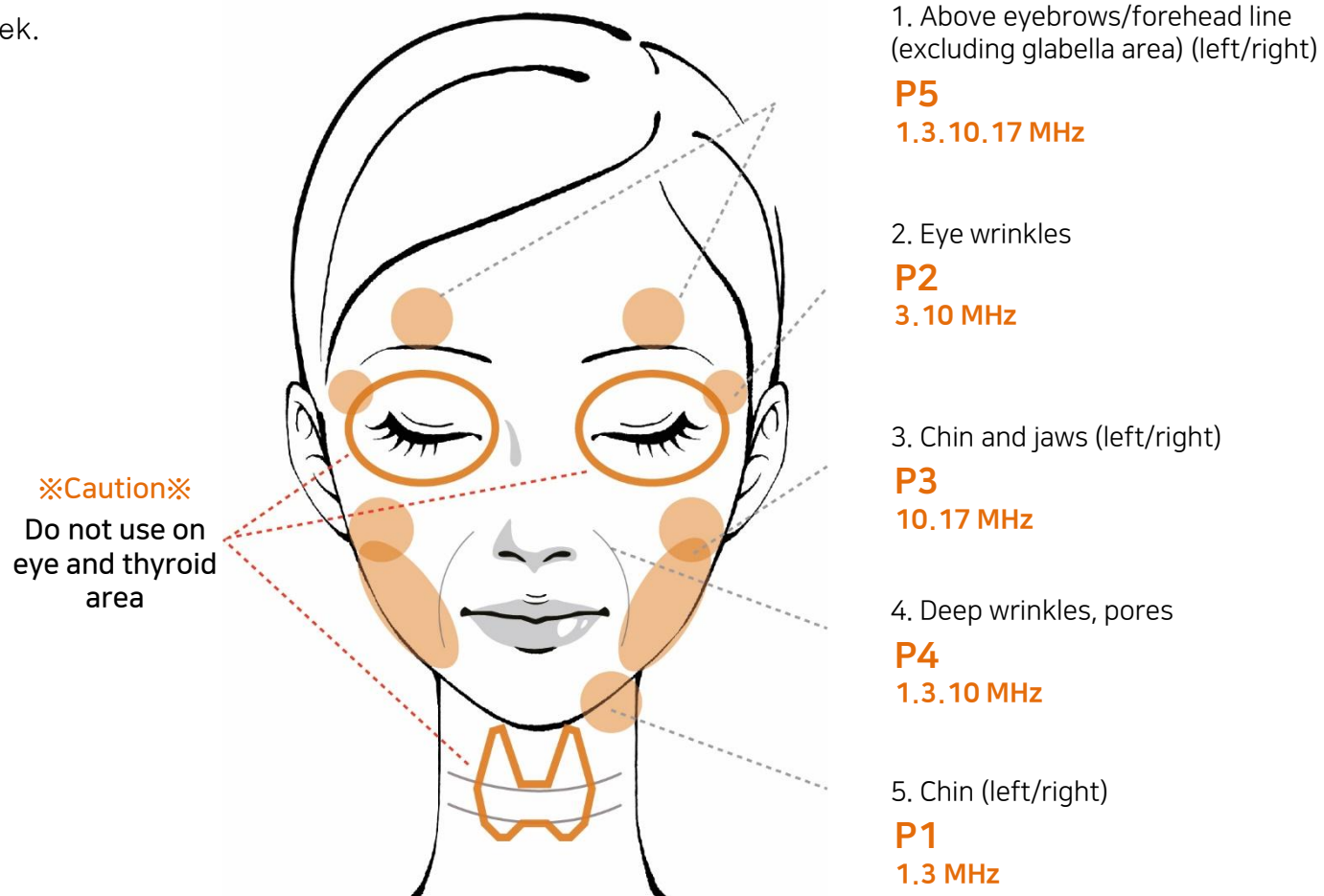




Clinical Trial Results

- Testing Institute : Cutis Biomedical Research Center Co., Ltd.
 - Tested Product : ALLFIT
 - Test Period : 2021.10.20 ~ 2021.10.22
 - Test Participants : 22 Female participants between the age of 30 to 59
- * Effects and results may vary depending on individuals
- 

1) Lifting (Skin Elasticity)


Skin Elasticity Measurement Results

There was an average **8.654% increase** from 0.628 before to 0.681 after one treatment.

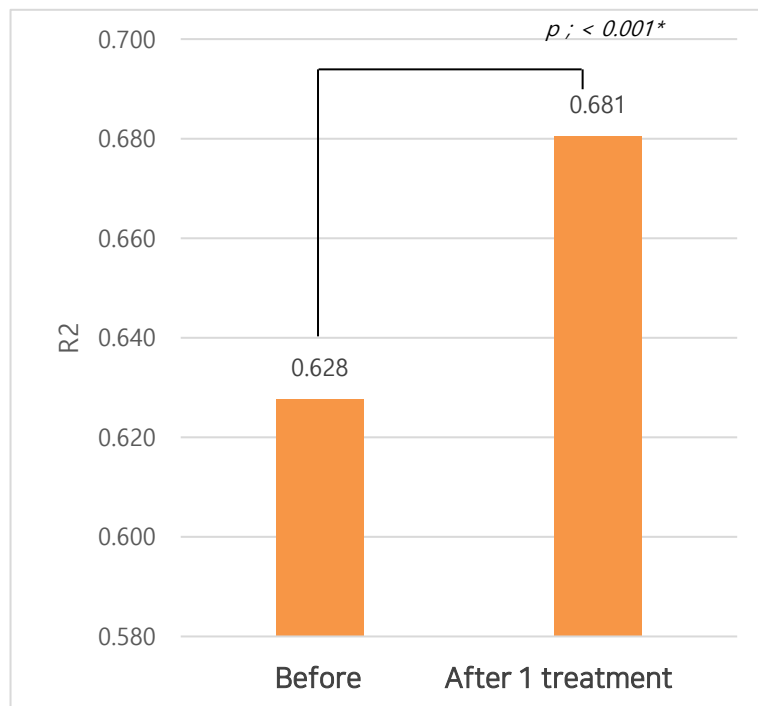

* ; $p < 0.05$ by Wilcoxon Signed Rank Test

<Change in skin elasticity>

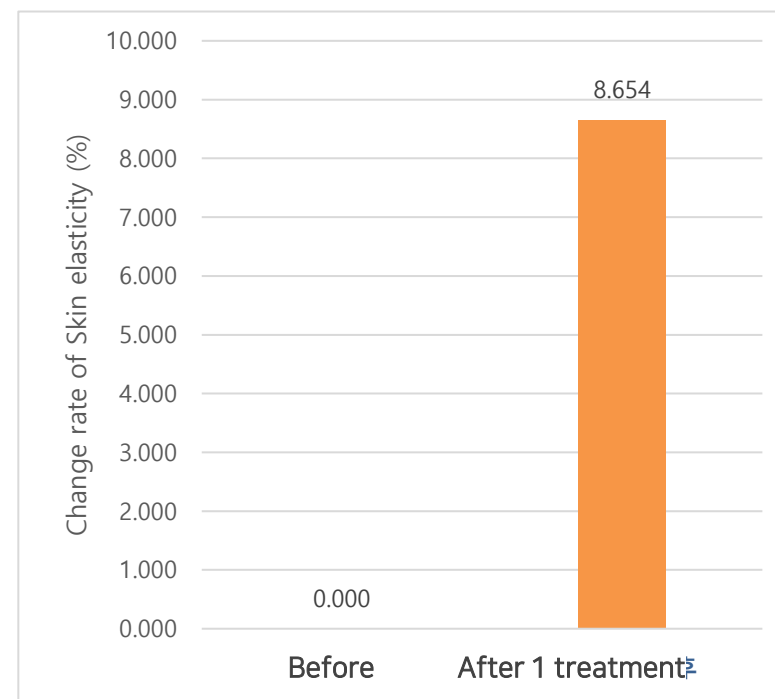


※ Rate of change (%) = (After - Before) / Before * 100

<Change in skin elasticity (%)>


1) Lifting (Skin Elasticity)

Skin Elasticity Measurement Results


There was an average **8.654% increase** from 0.628 before to 0.681 after one treatment.

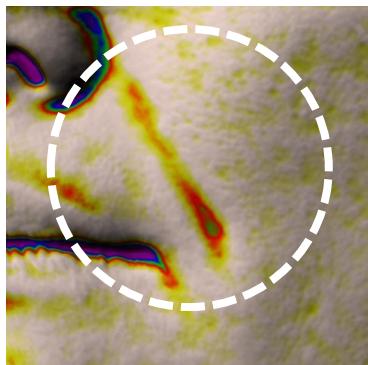


Before

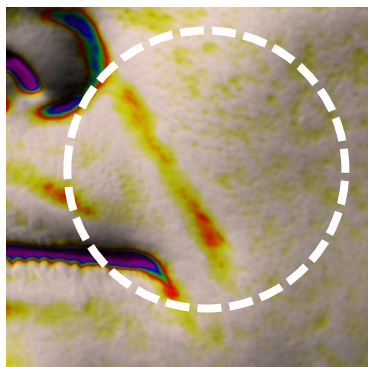


After 1 treatment

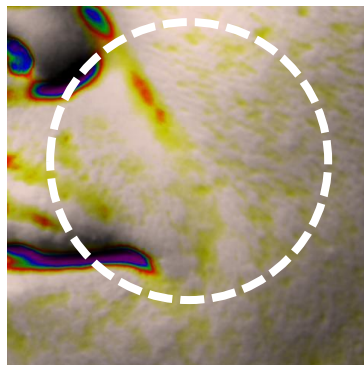


Before

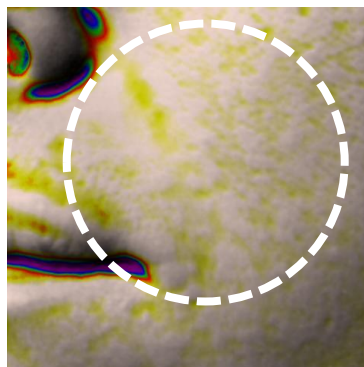

After 1 treatment

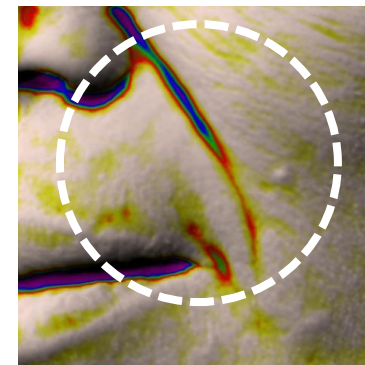

Before

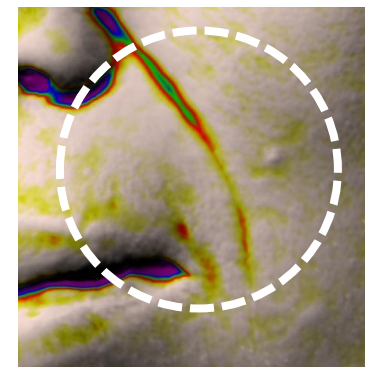


After 1 treatment

2) Hydration (Dermis Hydration)


Dermis Hydration Measurement Results

There was an average **2.843% increase** from 35.047 before to 36.041 after treatment.

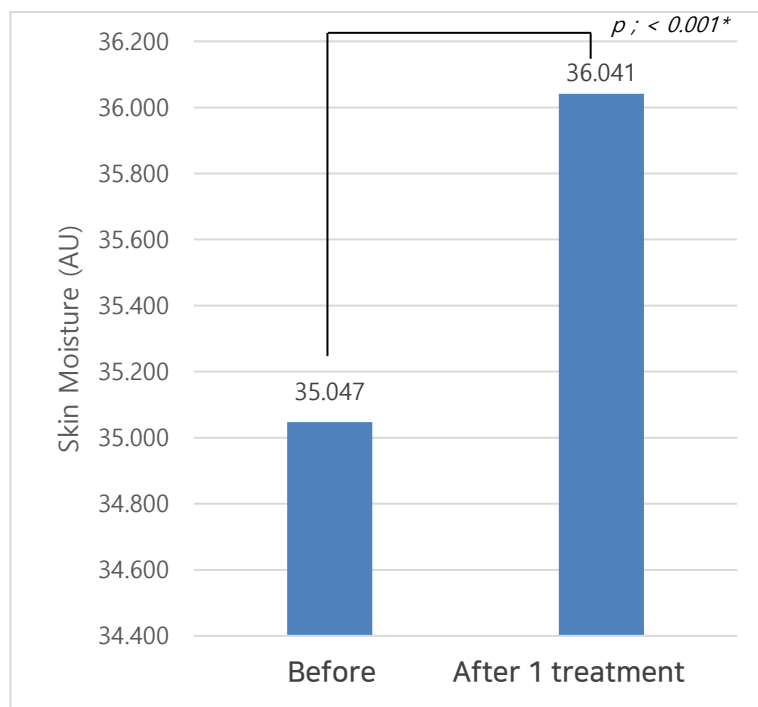

* ; $p < 0.05$ by Wilcoxon Signed Rank Test

<Change in dermis hydration>

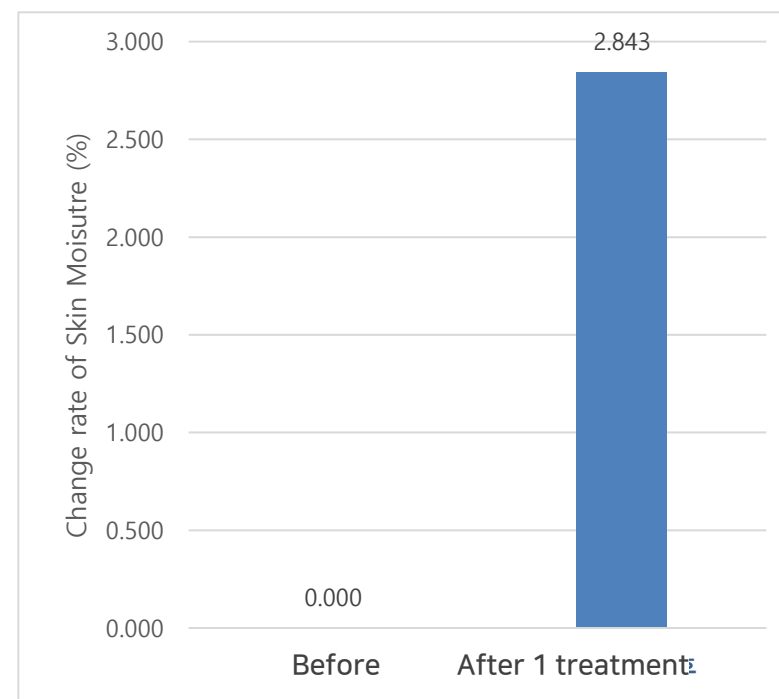

※ Rate of change (%) = (After - Before) / Before * 100

<Change in dermis hydration (%)>


2) Hydration (Dermis Hydration)

Dermis Hydration Measurement Results


There was an average **2.843% increase** from 35.047 before to 36.041 after treatment.

[Female / Age: 44 / Treatment order:

Ceutosome Fusion Shot → OXY-FX → OXY-FU]

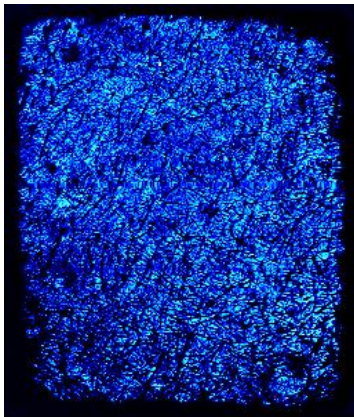


Before

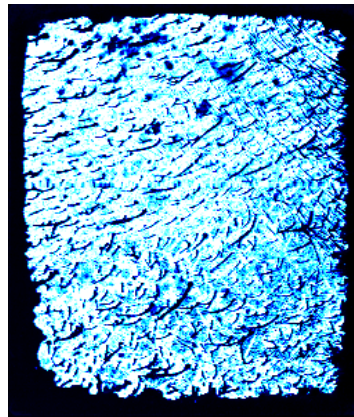


After 1 treatment

[Female / Age: 46 / Treatment order:

Ceutosome Fusion Shot → OXY-FX → OXY-FU]

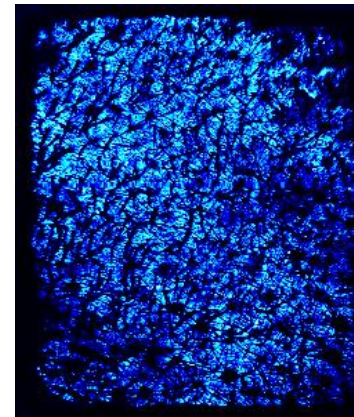

Before

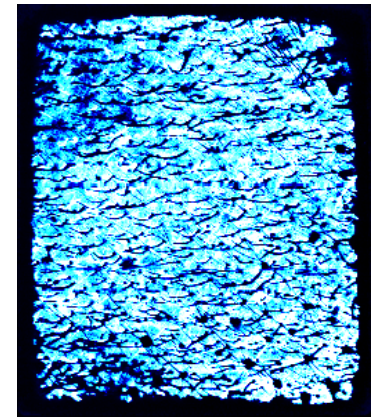

After 1 treatment


3) Brightening (Skin Radiance)

Skin Radiance Measurement Result

There was an average **3.324% increase** from 171.883 before to 177.592 after treatment.


Skin Brightness Measurement Result

There was an average **1.379% increase** from 71.533 before to 72.515 after treatment.

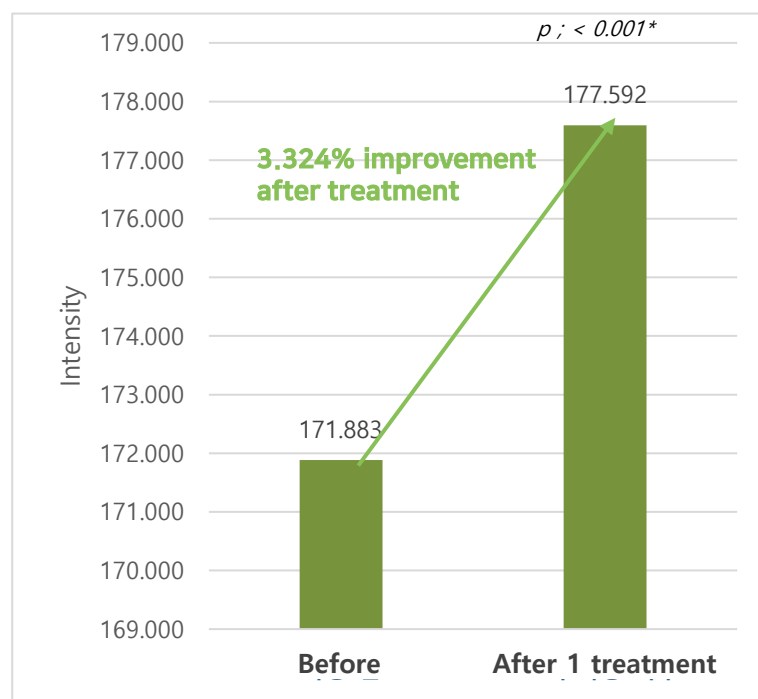

* ; $p < 0.05$ by Wilcoxon Signed Rank Test

<Change in skin radiation>

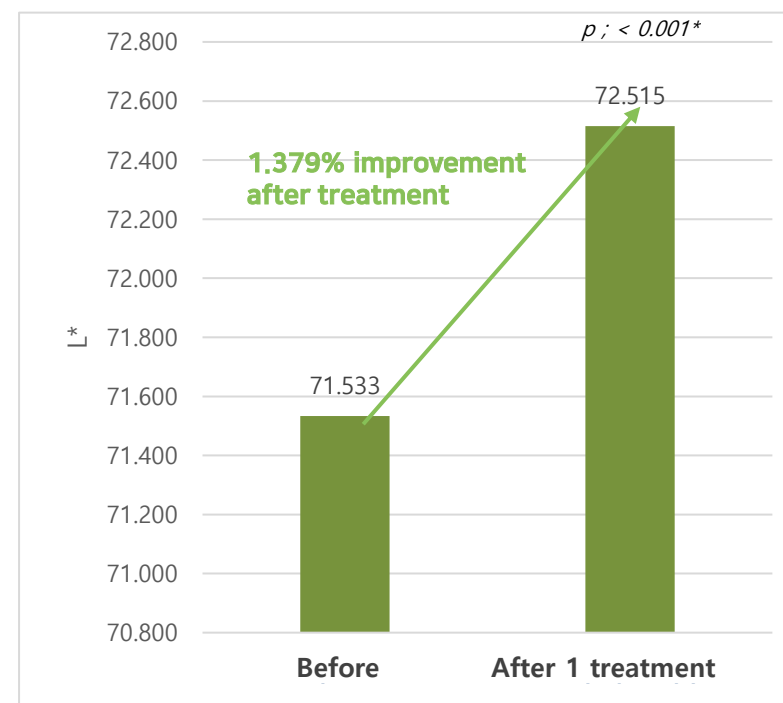

* ; $p < 0.05$ by Paired T-Test

<Change in skin brightness>

3) Brightening (Skin Radiance)

Skin Radiance Measurement Result

There was an average **3.324% increase** from 171.883 before to 177.592 after treatment.

Skin Brightness Measurement Result

There was an average **1.379% increase** from 71.533 before to 72.515 after treatment.

[Female / Age: 44 / Treatment order: Ceutisome Fusion Shot → OXY-FX → OXY-FU]



Before



After 1 treatment
(Skin Brightness)



After 1 treatment
(Skin Radiance)

Oxygen Waterful Lifting Program **By Skin Type**

ALLFIT

▪ **Anti-aging & Deep wrinkles**

Ceutisome W Fusion Shot → Contour V Gel → OXY-FX(P4) → OXY-FU → ADF → Oxygen Modeling Mask

▪ **Acne & Combination Skin**

- Ceutisome O Fusion Shot → Contour V Gel → OXY-FX(P2) → OXY-FU → ADF → Oxygen Modeling Mask
- Herbal Fluid → Contour V Gel → OXY-FX(P2) → ADF → Oxygen Modeling Mask

▪ **Hydration & Water glow care**

Ceutisome H Fusion Shot → Contour V Gel → OXY-FX(P3) → OXY-FU → ADF → Oxygen Modeling Mask

▪ **Hyperpigmentation & Brightening**

Ceutisome P Fusion Shot → Contour V Gel → OXY-FX(P3) → OXY-FU → ADF → Oxygen Modeling Mask

Combine with MTS therapy

Ceutisome Ampoule (by skin type) → **MTS** → Contour V Gel → OXY-FX → OXY-FU → ADF
→ Oxygen Modeling Mask

Home Care

D:O2 Activator → Moisture Aqua Serum → Skin Barrier ATO Cream



ALLFIT Recommended Skin Type

Recommended for people...

- ✓ with accentuated smile lines due to skin sagging
- ✓ with sagging skin below chin or jawline
- ✓ with saggy skin and fat layer that botox cannot undo
- ✓ who want skin lifting without surgical procedures
- ✓ who want treatment that requires no recovery period

Before & After



Acne Skin before/after



Facial Lifting before/after